

**RII – IV CONTRAST NECK CHEST ABDOMEN PELVIS
SIEMENS DEFINITION AS+ PROTOCOL**

Indications - mass, lymphoma, adenopathy, mets, abscess.

Position/Landmark	Head first or feet first-Supine 1cm superior to skull vertex																																			
Topogram Direction	Craniocaudal / Craniocaudal																																			
Respiratory Phase	Inspiration																																			
Scan Type	Helical																																			
Ref kV/Ref mAs/Rotation time (sec) Pitch / Speed (mm/rotation) Safire Strength / Dose Optimization	<table border="0" style="width:100%"> <tr> <td style="width:50%">Neck</td> <td style="width:50%">Chest Abd Pelvis</td> </tr> <tr> <td>Care kV 120/Care Dose4D 100/0.5sec</td> <td>Care kV 120/Care Dose4D 210/0.5sec</td> </tr> <tr> <td>.8:1 , 32.00mm</td> <td>.8:1 , 32.00mm</td> </tr> <tr> <td>3 / 7</td> <td>3 / 6</td> </tr> </table>	Neck	Chest Abd Pelvis	Care kV 120/Care Dose4D 100/0.5sec	Care kV 120/Care Dose4D 210/0.5sec	.8:1 , 32.00mm	.8:1 , 32.00mm	3 / 7	3 / 6																											
Neck	Chest Abd Pelvis																																			
Care kV 120/Care Dose4D 100/0.5sec	Care kV 120/Care Dose4D 210/0.5sec																																			
.8:1 , 32.00mm	.8:1 , 32.00mm																																			
3 / 7	3 / 6																																			
Detector width x Rows = Beam Collimation	0.625mm x 64 = 40mm (128 x .6mm)																																			
Average Tube Output	<table border="0" style="width:100%"> <tr> <td style="width:50%">First Helical: ctdi – 9.7mGy dlp – 305.6 mGy.cm</td> <td style="width:50%">Second Helical: ctdi – 11.0mGy dlp – 856 mGy.cm</td> </tr> </table>	First Helical: ctdi – 9.7mGy dlp – 305.6 mGy.cm	Second Helical: ctdi – 11.0mGy dlp – 856 mGy.cm																																	
First Helical: ctdi – 9.7mGy dlp – 305.6 mGy.cm	Second Helical: ctdi – 11.0mGy dlp – 856 mGy.cm																																			
First Helical Set Slice Thickness/ Spacing Algorithm Recon Destination	<table border="0" style="width:100%"> <tr> <td style="width:10%">recon</td> <td style="width:20%">body part</td> <td style="width:20%">thickness/ spacing</td> <td style="width:20%">algorithm</td> <td style="width:30%">recon destination</td> </tr> <tr> <td>1</td> <td>axial iv neck</td> <td>3mm x 3mm</td> <td>I40f medium</td> <td>pacs</td> </tr> <tr> <td>2</td> <td>coronal iv neck</td> <td>3mm x 3mm</td> <td>I40f medium</td> <td>pacs</td> </tr> <tr> <td>3</td> <td>thin neck</td> <td>.75mm x .7mm</td> <td>I40f medium</td> <td>terarecon</td> </tr> </table>	recon	body part	thickness/ spacing	algorithm	recon destination	1	axial iv neck	3mm x 3mm	I40f medium	pacs	2	coronal iv neck	3mm x 3mm	I40f medium	pacs	3	thin neck	.75mm x .7mm	I40f medium	terarecon															
recon	body part	thickness/ spacing	algorithm	recon destination																																
1	axial iv neck	3mm x 3mm	I40f medium	pacs																																
2	coronal iv neck	3mm x 3mm	I40f medium	pacs																																
3	thin neck	.75mm x .7mm	I40f medium	terarecon																																
Second Helical Set Slice Thickness/ Spacing Algorithm Recon Destination	<table border="0" style="width:100%"> <tr> <td style="width:10%">recon</td> <td style="width:20%">body part</td> <td style="width:20%">thickness/ spacing</td> <td style="width:20%">algorithm</td> <td style="width:30%">recon destination</td> </tr> <tr> <td>1</td> <td>axial chest</td> <td>5mm x 5mm</td> <td>I40f medium</td> <td>planning</td> </tr> <tr> <td>2</td> <td>axial abd pelvis</td> <td>5mm x 5mm</td> <td>I40f medium</td> <td>planning</td> </tr> <tr> <td>3</td> <td>axial chest abd pelvis</td> <td>5mm x 5mm</td> <td>I40f medium</td> <td>pacs</td> </tr> <tr> <td>4</td> <td>coronal chest abd pelvis</td> <td>5mm x 5mm</td> <td>I40f medium</td> <td>pacs</td> </tr> <tr> <td>5</td> <td>thin chest abd pelvis</td> <td>.75mm x .7mm</td> <td>I40f medium</td> <td>terarecon</td> </tr> <tr> <td>6</td> <td>lungs</td> <td>5mm x 5mm</td> <td>I70f very sharp</td> <td>pacs</td> </tr> </table>	recon	body part	thickness/ spacing	algorithm	recon destination	1	axial chest	5mm x 5mm	I40f medium	planning	2	axial abd pelvis	5mm x 5mm	I40f medium	planning	3	axial chest abd pelvis	5mm x 5mm	I40f medium	pacs	4	coronal chest abd pelvis	5mm x 5mm	I40f medium	pacs	5	thin chest abd pelvis	.75mm x .7mm	I40f medium	terarecon	6	lungs	5mm x 5mm	I70f very sharp	pacs
recon	body part	thickness/ spacing	algorithm	recon destination																																
1	axial chest	5mm x 5mm	I40f medium	planning																																
2	axial abd pelvis	5mm x 5mm	I40f medium	planning																																
3	axial chest abd pelvis	5mm x 5mm	I40f medium	pacs																																
4	coronal chest abd pelvis	5mm x 5mm	I40f medium	pacs																																
5	thin chest abd pelvis	.75mm x .7mm	I40f medium	terarecon																																
6	lungs	5mm x 5mm	I70f very sharp	pacs																																
Scan Start / End Locations DFOV	<table border="0" style="width:100%"> <tr> <td style="width:50%">neck external auditory meatus aortic arch 18cm</td> <td style="width:50%">chest abd pelvis 1cm superior to lung apices lesser trochanters 38cm</td> </tr> </table>	neck external auditory meatus aortic arch 18cm	chest abd pelvis 1cm superior to lung apices lesser trochanters 38cm																																	
neck external auditory meatus aortic arch 18cm	chest abd pelvis 1cm superior to lung apices lesser trochanters 38cm																																			
IV Contrast Volume / Type / Rate	30mL Iohexol (Omnipaque 300) followed by 40mL of saline prior to scouts then 5 minute delay then 100mL Iohexol (Omnipaque 300) , 3mL/sec When oral contrast is prescribed, refer to the appropriate oral contrast agent’s preparation and procedure guide.																																			
Scan Delay	30 seconds																																			
2D/3D Technique Used	Workstream 4D mpr of 3mm x 3mm coronal neck series, auto-transferred to PACS. Workstream 4D mpr of 5mm x 5mm coronal chest abd pelvis series, auto-transferred to PACS.																																			
Comments: Recons 1 and 2 of the chest abdomen pelvis are for planning only. This is needed to apply X-Care to only the chest; while keeping the chest abdomen and pelvis in a single volume. Recon 3 is a thin helical volume of the neck that is archived to the TeraRecon server. Recon 5 is a thin helical volume of the chest abd pelvis that is archived to the TeraRecon server.																																				
Images required in PACS	Topograms, 3mm x 3mm neck, 3mm x 3mm coronal neck, 5mm x 5mm axial chest abd pelvis, 5mm x 5mm coronal chest abd pelvis, 5mm x 5mm axial lungs, Patient Protocol																																			